

Procedure to Sterilize Aluminum Bur Block

1. Sterilizers (Dry Heat, Autoclave and Chemiclave)

Operate sterilizer unit at 121 °C high-pressure saturated steam for 15 – 20 minutes.

2. Ultrasonic Cleaning

Do not use ultrasonic solutions containing alkali products such as sodium hydroxide, potassium hydroxide, or hydroxides.

- Always mix solutions according to instructions.
- A five-minute cycle time is recommended.
- Remove the burs from ultrasonic and rinse well with water.
- Dry the bur block thoroughly before placing in sterilization pouches.



Name	Brown Cup	Green Cup	White Cup	Round Bur	Tapered Bur	Football Bur
Item No.	2000989	2000988	2000990	2000987-018	2000985-021	2000986-023
Shank	RA	RA	RA	FG	FG	FG
Size				018	021	023

3007394_01 GD-458-0912

BruxZir™

ADJUSTMENT & POLISHING KIT



DISTRIBUTED BY:

GLIDEWELL DIRECT

CLINICAL AND LABORATORY PRODUCTS

2181 Dupont Drive • Irvine, CA 92612
(888) 303-3975 • www.glidewelldirect.com

Always use light pressure with water and air spray when adjusting BruxZir® restorations to avoid micro-fractures.



A football-shaped bur is most effective for adjusting occlusion on the occlusal surfaces of posterior teeth and lingual surfaces of anterior teeth.



A tapered bur is most effective for adjusting cusps or proximal contacts.



A round bur is used to adjust a cusp fossa and for creating endodontic access.

Glidewell Direct (888) 303-3975 • www.glidewelldental.com



Using light pressure and no water, begin pre-polishing with the brown cup to remove abrasions left by the diamonds during adjustments. The brown cup can also be used alone for minor adjustments if you don't want to use a diamond.



Continue pre-polishing with the green cup until a more glossy look starts to appear on the adjustment areas.



Finally, use the white cup with light to medium pressure to achieve a "wet" high shine.